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Produkt: Carepants

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# Instructions for Use

Product: Carepants

Manufacturer: Caresana GmbH, Offenburg, Germany

Risk Class: Class I (non-sterile)

## 1. Name and Intended Purpose

Carepants are absorbent incontinence pants for adults with moderate to severe urinary incontinence. They are designed to absorb and securely contain urine, helping to prevent skin irritation, leakage, and odor. The product is intended for single use only.

## 2. Target Group

- Adults with moderate to severe urinary incontinence
- Mobile or partially mobile users
- Also suitable for caregivers
- Also suitable for mild fecal incontinence

## 3. Instructions for Use

Putting on:

1. Remove product from packaging.
2. Ensure absorbent core is centered.
3. Put on like regular underwear.
4. Ensure proper fit.

Changing:

1. Tear pants open at the sides.
2. Dispose of in household waste – do not flush.
3. Clean skin and put on new pants.

## 4. Sizes

S: 50–76 cm

M: 70–110 cm

L: 105–150 cm

XL: 145–170 cm

## 5. Warnings

- For single use only
- Do not use if packaging is damaged
- Consult a physician if irritation occurs
- Not suitable for severe fecal incontinence

## 6. Storage and Disposal

- Store in a cool, dry, and dark place
- Keep away from heat sources
- Dispose of in household waste

## 7. Materials

- Top layer: hydrophilic nonwoven
- Absorbent core: fluff pulp, SAP
- Back sheet: breathable PE film
- Elastic: latex-free

## 8. Symbols on Packaging

Single use only		Manufacturer	
Consult instructions for use		Medical Device	
Do not throw in the toilet!		Unique Device Identification	
Dispose of with household waste		Conformité Européenne European conformity	

## 9. Manufacturer Information

Caresana GmbH  
Am alten Schlachthof 14  
77652 Offenburg, Germany  
Tel.: +49 781 953 80 600  
Email: info@caresana.eu  
www.caresana.de

## 10. Reporting Adverse Events

Please report any adverse events to the manufacturer or competent authority.

## 11. Last Revision

Date: August 2025  
Version: 1.0